


Unides® AF wipes

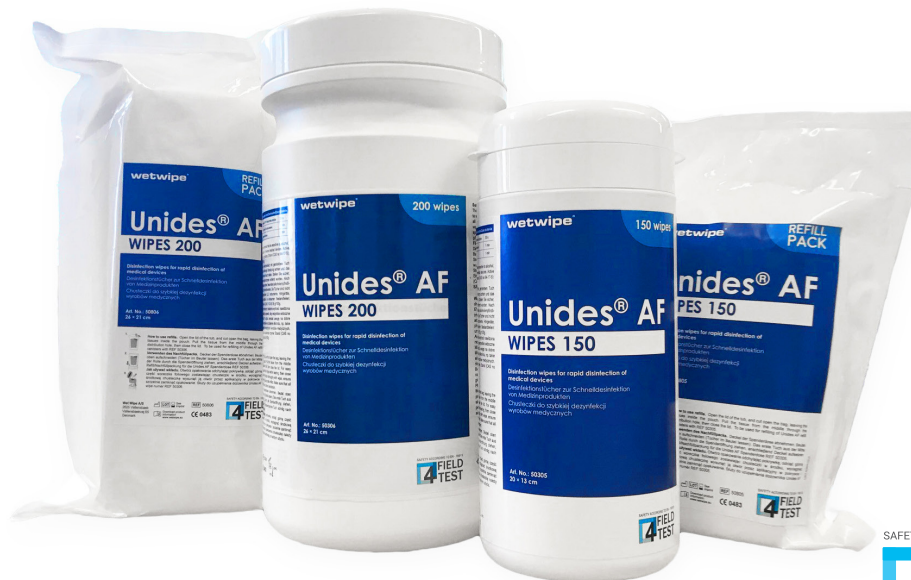

SAFETY ACCORDING TO EN - 16615
4 FIELD TEST

Ready to use rapid disinfection wipes based on alcohols for residue free disinfection. Broad antimicrobial spectrum and short drying time. Fragrance free.

Use areas

Use for disinfection of wipeable surfaces and non-invasive alcohol resistant medical devices.

Application

Wipe surface and allow disinfectant solution to take effect. Keep surface wet for the whole exposure time.

Composition

Active ingredient solution contains: 24 g Ethanol and 37 g Propan-1-ol.

Antimicrobial Spectrum

Active ingredient solution is:

- Bactericidal (incl. MRSA)
- Mycobactericidal (M. terrae; M. avium)
- Yeasticidal, fungicidal (C. albicans; A. brasiliensis)
- Virus inactivating (HBV/HIV, HCV, BVDV, Rota-; Adeno-, Noro-, Vaccinia-, Influenza- and Herpes viruses)

Notifications/registrations

VAH List of Disinfectants

IHO notification

BAuA registered according to EU Biocidal Regulation and ECHA guidelines.

CE certification

Certified as Medical Device class IIa according to EU Medical Device Directive 93/42/EEC

Chemical-physical data

Color: colourless
pH-value: 6-8
Flash point: 25°C DIN 51755

Special advice

Use disinfectants safely. Always read the label and product information before use.

4- Field Test

Tested in the 4-field test according to EN 16615 and VAH guidelines by Hygieneinstitut IKI - PD Dr.Pitten, Gießen.

IHO BAuA Reg.
Nr.: N-72472

Wet Wipe A/S
info@wetwipe.eu
Tel: +45 70 266 244
www.wetwipe.eu

CVR/VAT no: DK 30207726

Wet Wipe GmbH
info@wetwipe.de
Tel: +49-6227-8724000
www.wetwipe.de

Ust-IdNr. DE 178836011

52-050, EN Ver. 2
Aug 2020

wetwipe®
Developed in Denmark
Made in Germany

Exposure times

	EN test	Exposure times
Bacteria	EN 13727, EN 14561, EN 16615, DGHM/VAH	30 sec
<i>S. aureus (MRSA)</i>	DGHM/VAH	30 sec
Yeast	EN 13624 DGHM/VAH	30 sec
<i>Mycobacteria</i>	EN 14348	60 sec
Fungus	EN 13624 DGHM/VAH	60 sec
HIV/HBV,HCV,Corona	EN 14476	30 sec
Influenza / Herpes	EN 14476	30 sec
BVDV	EN 14476	30 sec
Vaccinia	EN 14476	15 sec
Rotavirus	EN 14476	60 sec
Norovirus	EN 14476	60 sec

Ordering information

	Art.-Nr.	Quantity	VE	PZN
Unides AF				
Tub 13 x 20 cm , white	50305	150 St	12 Tubs	13654648
Refill 13 x 20 cm , white	50805	150 St	12 Refills	13654648
Unides AF Jumbo				
Tub 21 x 26 cm, white	50306	200 St	6 Tubs	13654660
Refill 21 x 26 cm, white	50806	200 St	6 Refills	13654677